CAD Ref: PLSS146

Dimensions: 165 x 127 x 165mm


Ref no: 47648-01
Created by: Natalie Whiteside

Refer to ARS PD-531897 and Master Text IFU PD-572948

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Colour information

- PMS 361
- PMS Cool Grey 10
- Spot color PMS 485 (485) and PMS 486 (486) Dimensions 165mm x 165mm x 127mm

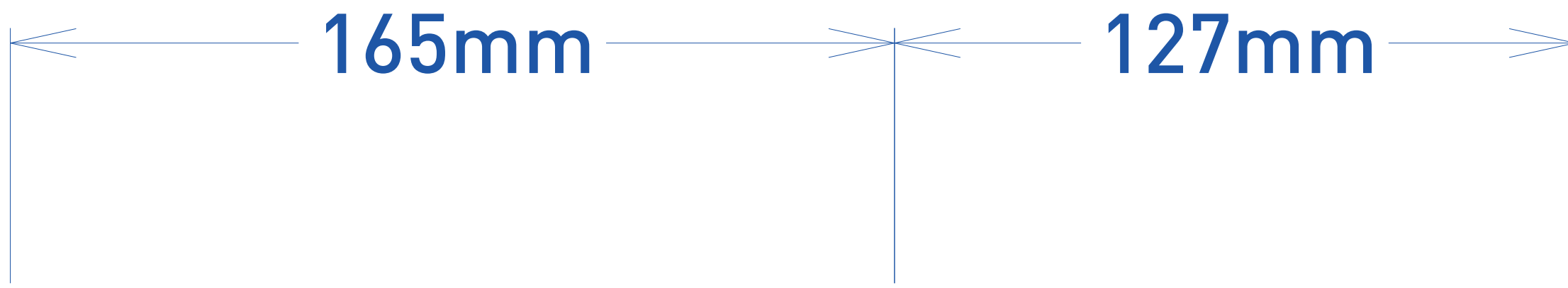Mölnlycke®

Tubigrip®

Elasticated tubular support bandage

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Mölnlycke Health Care Group of Companies.
The Mölnlycke and Tubigrip trademarks, names and logos are registered globally to one or more of the
Manufacturers / Fabricants

Elasticated Tubular Bandage BP Type A. Instructions for use: Consult size guide. Cut the bandage to twice the length of the affected limb, if required make 2 slots for a thumb hole. Pull over the limb, then double back. Dispose as should be handled according to local environmental procedures.

Precautions: Do not use Tubigrip if patient/user has known hypersensitivity to the fibre content. If your condition worsens stop use and consult a health care professional. Keep clean, if soiled discard and replace bandage. If any serious incident has occurred in relation to the use of Tubigrip, it should be reported to Mölnlycke Health Care. Fibre Content: Cotton, Elastodiene and Polyester.

Caution: This product contains natural rubber latex which may cause allergic reactions, including anaphylactic responses.

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Mölnlycke®

Tubigrip®

Elasticated tubular support bandage

10m

1 x 10m / 10.9 yards

USA 1-800-882-4582
Canada 1-800-494-5134

Australian Sponsor Address:
Mölnlycke Health Care Pty. Ltd.
12 Narabang Way
Belrose
NSW 2085
Australia

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