Mölnlycke® Mepi™ Debripad

Mepi™ Debripad

Microfiber pad for wound bed preparation

INSTRUCTIONS FOR USE

Intended Purpose
Mepi™ Debripad is intended for the debridement of superficial, chronic and acute wounds and the skin surrounding the wound. It is to be used to remove devitalized tissue from the wound bed, and to absorb exudate and debris during debridement. It is suitable for dry, paronchymalike, hyperkeratotic, seborrheic or only healthy or damaged skin.

Product Description
Mepi™ Debripad is a polyester/polymide microfiber pad designed for fast, safe and low pain debridement of wounds. Each pad is provided sterile and is for single use only.

Not made with natural rubber latex

Indications
Mepi™ Debripad is indicated for use on diabetic ulcers, arterial and venous ulcers, pressure ulcers, postoperative wounds healing by secondary intention, traumatic or surgical wounds (e.g. abrasions, incisions, lacerations), and, burns and scalds (3rd degree: following surgical debridement).

Contraindications

- Do not use on patients with a known sensitivity to any of the composition materials.
- Surgical implantation.
- Do not use on hard necrotic tissue or viable/granulating tissue.

Precautions or Warnings

- For use under the guidance of a healthcare professional.
- Single use only, do not re-use in whole or in part, as it may compromise sterility and/or the performance of the pad.
- Do not re-sterilise.
- Do not use if the package is damaged or if there are signs of contamination on the pad.
- Mepi™ Debripad must not be ingested and must be kept away from children and animals.
- Do not use if primary packaging is damaged or unintentionally opened before use.
- For disposal of the device please refer to local guidance on the disposal of clinical waste.

DIRECTIONS FOR USE

- Remove any wound dressing.
- Debride until healthy tissue is visible.
- Soak Mepi™ Debripad in 20ml sterile saline solution or in other suitable wound cleansing solution.
- Insert hand into the mitt of the pad.
- Wipe the wound surface gently without pressure using the soft side of the pad that faces away from the palm of the hand.
- If required cleanse the peri wound skin using another thoroughly moistened Mepi™ Debripad.
- Following use, standard treatment for the condition should be continued.
- Dispose of the used Mepi™ Debripad in line with local procedures for clinical waste.

When using the Debripad pad standard practice should be followed, including the use of an aseptic technique, and sterile surgical gloves.

1. Remove any wound dressing.
2. Debride until healthy tissue is visible.
3. Soak Mepi™ Debripad in 20ml sterile saline solution or in other suitable wound cleansing solution.
4. Insert hand into the mitt of the pad.
5. Wipe the wound surface gently without pressure using the soft side of the pad that faces away from the palm of the hand.
6. If required cleanse the peri wound skin using another thoroughly moistened Mepi™ Debripad.
7. Following use, standard treatment for the condition should be continued.
8. Dispose of the used Mepi™ Debripad in line with local procedures for clinical waste.

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